FISIO CHAMBER
vision **BPA FREE**

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Koo Medical Equipment (Shanghai) Co., Ltd
100 Zhongde Road, Dakun Industrial Park, Songjiang Shanghai, 201614, P.R. CHINA

 Inhalation Vision Valve • Universal Adapter
“Proper Fit” • Spiral Vents • Mask Comfort & Seal
with Exhalation Safety Valve

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In order to keep the full efficiency of the device, we advise to change the valve block every 12 months (ref. KM-1028)

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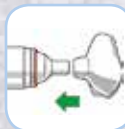
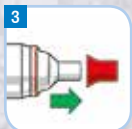
instructions for use

1 Before use, carefully check all the parts, valves and holes.
Replace if any defect is noticed.

2 Remove cap from MDI (Metered Dose Inhaler).
Shake the MDI per instructions.
Insert and seal the MDI into the universal adapter "Proper Fit".

or

Remove cap from Chamber.

Insert the F시오 Mask into mouthpiece of the chamber.
Verify the mask safety valve is openable (if not replace it).
Apply the mask to the face. Verify to have properly sealed.
OR

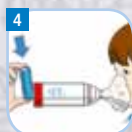
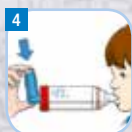
If the patient can cooperate, apply the mouthpiece only into the mouth and verify to have properly sealed.

Inhale and exhale normally.

Then press the MDI one time and start to inhale slowly.
Carefully look at the **VISION VALVE** as the movement shows the inhalation act.

Maintain seal the mask or the mouthpiece for at least 5 inhalation acts after MDI is pressed.

Wait before a new puff and repeat this step as for doctor or MDI instructions. Always follow prescribed dosage.

or

vision valve

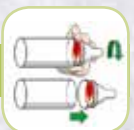
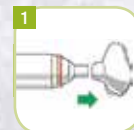
CAUTIONS

Follow instructions before every use. Do not leave F시오 Chamber Vision or part of it unattended with children, as some components are small enough to be swallowed. This product is a Single Patient Use. Do not use with different patients to prevent contaminations. The product should be replaced after max. 2 years of continual use. Replace the product if damage, sealing of the valve, environmental condition, storage and frequency of use make the product different from the original use. Dispose according regulations.

The use and the sale of F시오 Chamber Vision is restricted by or on the order of physician. Monitor the patient during use.

cleaning instructions


Clean before first use, then at least once a week following instructions.

1 Remove parts as shown and open the chamber.

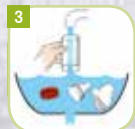2 Soak the parts for 15 minutes in lukewarm clean water and mild detergent.
Agitate gently.

3 Rinse the parts in clean water.
Shake out excess of water and allow to air dry.

Do not use dry clothes to dry.
Make sure all the parts are dry before reassembly.

Reassembly parts as shown and stock the product in a dry and clean container.

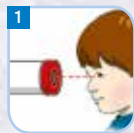
Storage Temperature $-10^{\circ} / +45^{\circ}\text{C}$
Relative Humidity 30% / 80%

- A** Anti-static chamber
- B** Inhalation Vision Valve
- C** Universal adapter "Proper Fit" with spiral vents
- D** Exhalation port (DO NOT OCCLUDE)
- E** Mask Comfort & Seal with exhalation safety valve

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